VetExpert Rapid Feline Toxoplasma Ab

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VetExpert Rapid Feline Toxoplasma Ab Test Kit

• Principles

The VetExpert Rapid Feline Toxoplasma Ab Test Kit is a chromatographic immunoassay for the qualitative detection of antibody against *Toxoplasma gondii* in feline serum, plasma or whole blood. The Antigen Rapid Feline Toxoplasma Ab Test Kit has the letter "T" and "C" as test line and control line on the surface of the device. Both the test line and control line in the result window are not visible before applying any samples. The control line is used for procedural control, and should always appear if the test procedure is performed properly and the test reagents of the control line are working. A purple test line will be visible in the result window if there are enough antibodies against *T. gondii* in the specimen.

The specially selected recombinant *T. gondii* antigen is used in the test band as a capture material. These enable the VetExpert Rapid Feline Toxoplasma Ab Test Kit to identify antibody against *T. gondii* in feline serum, plasma or whole blood with a high degree of accuracy.

• Materials provided (5/10 Tests/Kit)

- 1) Ten (5/10) VetExpert Rapid Feline Toxoplasma Ab Test Devices
- 2) One (1) Assay diluents (3 ml)
- 3) Ten (5/10) Disposable capillary tubes
- 4) Ten (5/10) Anti-coagulant bottles
- 5) One (1) Instruction for use
- ◆ A dark color score line on the capillary tube is the indicator line for 10 µl.

• Precautions

- 1) For veterinary diagnostic use only.
- 2) This kit is for only cat. Do not apply to other animals.
- 3) For the best results, strict adherence to the instructions is required.
- 4) Do not open or remove test kit from their individually sealed pouches until immediately before use.
- 5) Do not use the test kit if the pouch is damaged or the seal is broken.
- 6) Do not reuse test kit.
- 7) All reagents must be at room temperature before running assay.
- 8) Do not use reagents beyond the stated expiration date marked on the label.
- 9) The components in this kit have been quality control tested as standard batch unit. Do not mix components from different lot numbers.

• Storage and Stability

- 1) The kit can be stored at room temperature or refrigerated (2–30°C). DO NOT FREEZE.
- 2) Do not store the test kit in direct sunlight.
- 3) The test kit is stable through the expiration date marked on the package label.

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