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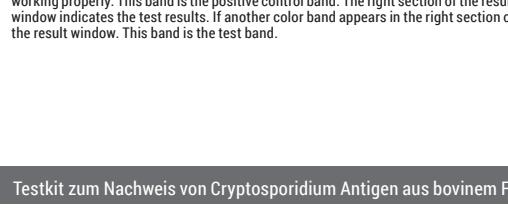
One Step Cryptosporidium Antigen Test

• Procedure of the Test

- 1) Collect the samples from feces using the sample collection swab.
- 2) Insert the sampling swab into the assay diluents tube.
- 3) Mix the sampling swab until the sample has been dissolved into the assay diluents.
- 4) Leave the tube until the large particles have settled down to the bottom of the tube. (Approximately 1 minute)
- 5) Remove the test devices from the foil pouch, and place it on a flat and dry surface.
- 6) Using a disposable dropper(provided), take an aliquot from the extracted and mixed sample in the tube.
- 7) Add 4(four) drops into the sample hole with the disposable dropper. The mixed assay diluents should be added exactly, slowly drop by drop.
- 8) As the test begins to work, you will see purple color move across the result window in the center of the test device. If the migration has not appeared after 1 minute, add one more drop of the mixed sample to the sample hole.
- 9) Interpret test results at 10 minutes. Do not decide after 20 minutes.

[Test Procedure]


[Interpretation of the Test]

A color band will appear in the left section of the result window to show that the test is working properly. This band is the positive control band. The right section of the result window indicates the test results; if another color band appears in the right section of the result window, this band is the test band.


1) Negative result

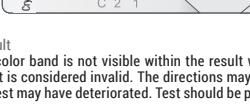Only one band (“T”) within the result window indicates negative against Cryptosporidium antigen.

2) Positive result

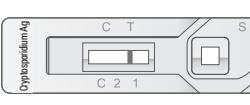Two color bands (“T” and “C”) within the result window indicate the presence of Cryptosporidium antigen.

3) Invalid Result

If the purple color band is not visible within the result window after performing the test, the result is considered invalid. The directions may not have been followed correctly or the test may have deteriorated. Test should be performed again.

• Remarks

- 1) This test is for in-vitro diagnostic use only.
- 2) This test detects the presence of Cryptosporidium antigen in the specimen and should not be used as the sole criterion for the diagnosis of them.
- 3) As with all diagnostic tests, all results must be considered with other clinical information available to the veterinarian.

VetExpert Rapid Cryptosporidium Ag

• Principles

VetExpert Rapid Cryptosporidium Ag Test is a solid phase immunochromatographic assay for the rapid, qualitative detection of Cryptosporidium antigen in calf feces. VetExpert Rapid Cryptosporidium Ag Test has the letters “T” and “C” as the Test line and Control line on the surface of the device. Both the test line and control line in the result window are not visible before applying any samples. The control line is used for procedural control, and should always appear if the test procedure is performed properly and the test reagents of the control line are working. A purple test line will be visible in the result window if there is enough antigen of each target in the specimen.

- Materials Provided (5 Tests/Kit)
- 1) 5 VetExpert Rapid Cryptosporidium Ag Test
- 2) 5 specimen tubes containing assay diluent buffer
- 3) 5 sample collection swabs
- 4) 5 disposable droppers
- 5) Instruction for use

• Precautions

- 1) For the best results, strict adherence to these instructions is required.
- 2) All specimens should be handled as being potentially infectious.
- 3) Do not open or remove test devices from their individually sealed pouches until immediately after their use.
- 4) Do not reuse test devices.
- 5) All reagents must be at room temperature before running the assay.
- 6) Do not use reagents beyond the stated expiration date marked on the package label.
- 7) The components in this kit have been quality control tested as a standard batch unit. Do not mix components from different lot numbers.
- 8) The assay diluents contain low concentration of sodium azide as a preservative. Sodium azide is toxic and should be handled carefully to avoid ingestion and skin contact.
- 9) Storage and Stability
- 10) Specimen Collection and Preparation
- 11) Test should be performed with the calf feces.
- 12) After collecting the specimen using swab, the specimen should be immediately extracted and tested.
- 13) If specimens are not immediately tested, they should be refrigerated at 2-8°C, for storage not less than 48 hours, freeze the specimen at -20°C or below.

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Hergestellt durch Bionote Inc.