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VetExpert Rapid FCoV Ab Test

● Principles

The VetExpert Rapid FCoV Ab Test is a chromatographic immunoassay for the qualitative detection of Feline Corona virus antibodies in feline whole blood, serum, or plasma. The VetExpert Rapid FCoV Ab Test has a letter of “T” and “C” as test line and control line on the surface of the device. Both the test line and control line in the result window are not visible before applying any samples. The control line is used for procedural control, and should always appear if the test procedure is performed correctly and the test reagents of the control line are working. A purple test line will be visible in the result window if there is sufficient FCoV antibody in the specimen. The specially selected Feline Corona Virus antibodies are used in the test band as both capture and detector materials. These enable the VetExpert Rapid FCoV Ab Test to identify Feline Corona virus antibodies in feline whole blood, serum, or plasma with a very high degree of accuracy.

● Materials provided (2, 5, 10 tests/kit)

1) 2, 5, 10 VetExpert Rapid FCoV Ab Tests

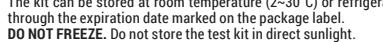
2) 1 assay diluents (2 ml)

3) 2, 5, 10 disposable capillary tube for specimen

4) Instruction for use

5) One(1) Instructions for use

◆ A dark color score line on the capillary tube is the indicator line for 10 µl

● Precautions

1) For veterinary diagnostic use only.

2) For best results, strict adherence to these instructions is required.

3) All specimens should be handled as being potentially infectious.

4) Do not open or remove test kit from their individually sealed pouches until immediately before their use.

5) Do not use the test kit if the pouch is damaged or the seal is broken.

6) Do not reuse test kit.

7) All reagents must be at room temperature before running the assay.

8) Do not use reagents beyond the stated expiration date marked on the label.

9) The components in this kit have been quality control tested as standard batch unit. Do not mix components from different lot numbers.

● Storage and Stability

The kit can be stored at room temperature (2–30°C) or refrigerated. The kit is stable through the expiration date marked on the package label.

DO NOT FREEZE. Do not store the test kit in direct sunlight.

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