VETEXPERT Bovine KG

VETEXPERT GLUCOSE TEST STRIPS

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VETEXPERT Bovine KG

VETEXPERT GLUCOSE TEST STRIPS

USE ONLY WITH VETEXPERT BOVINE KG CHECK MULTI-FUNCTIONAL MONITORING SYSTEM

For veterinary use only.

WARNINGS

- For *in vitro* diagnostic use.
- For veterinary use only.
- For single use only.
- Please read this sheet and your **VetExpert Bovine KG Check** Multi-Functional Monitoring System Owner's Manual before you use this test strip. Use only VetExpert Blood Glucose Test Strips with **VetExpert Bovine KG Check** Multi-Functional Monitoring System to obtain accurate results, and be covered by the manufacturer's warranty.
- Results may be inaccurate when testing on patients with abnormally low blood pressure or those who are in shock.
- Keep test strips and lancets away from small children. If swallowed, consult a doctor immediately for advice.

INTENDED USE

VetExpert Blood Glucose Test Strips, when used together with VetExpert Bovine KG Check Multi-Functional Monitoring System, allow blood glucose levels. It uses fresh capillary whole blood samples, and from venous whole blood. It can be used on calves. Veterinarians may use test strips to test capillary and venous blood sample.

LIMITATIONS

- Hematocrit: The hematocrit level is limited to between 0% and 70%.
- Use only heparin for anticoagulation of fresh capillary or venous whole blood.

STORAGE AND HANDLING

IMPORTANT: Do not use the test strips if they have expired.

- Test strips expire 6 months after first opening. Write the first opening date on the test strip vial when you first opened it.
- Store the test strips in a cool, dry place between 2°C and 30°C and below 90% relative humidity.
- Keep the test strips away from direct sunlight. Do not store the test strips in high humidity.
- Store the test strips in their original vial ONLY. Do not transfer them to a new vial or any other containers.
- Do not touch the test strips with wet hands.
- Use each test strip immediately after taking it out of the vial or individual foil packet. Close the vial immediately after taking out a strip.
- Keep the vial closed at all times.
- Do not bend, cut, or alter the test strip.

STRIP APPEARANCE

- Absorbent Hole**
Apply a drop of blood here. The blood will be automatically absorbed.
- Confirmation Window**
This is where you confirm if enough blood has been applied to the absorbent hole in the strip.
- Test Strip Handle**
Hold this part to insert the test strip into the slot.
- Contact Bars**
Insert this end of the test strip into your meter.
Push it in firmly until it will go no further.

ATTENTION:

The front side of the test strip should face up when inserting the test strip. Test results may be wrong if the contact bar is not fully inserted into the test slot.

TESTING BLOOD GLUCOSE
PLEASE WASH AND DRY YOUR HANDS BEFORE PERFORMING ANY TESTING.

STEP 1

Insert the test strip fully into the slot of the meter until it will go no further. When the strip is fully inserted, the meter will do several self-checks.

STEP 2

Collect a blood sample for about 0.8 µL with the test strip. A sufficient quantity of blood is required for the test to provide accurate results. Touch the blood drop with the absorbent hole of the test strip, and wait until the confirmation window is fully covered. Do NOT apply a smeared blood sample. The meter will start counting down.

STEP 3

After a few seconds, the meter will display your blood glucose level. The last reading will be automatically saved in the meter. Turn it off by removing the test strip and throw away the used test strip.

Please refer to your Owner's Manual for further information.

The used test strip are potentially biohazardous. Please dispose of them carefully according to your local regulations.

Reading Your Result

Blood glucose readings deliver plasma equivalent results and are displayed either in milligrams of glucose per deciliter of blood (mg/dL) or in millimoles of glucose per liter of blood (mmol/L). The measurement range of this meter is 10-700 mg/dL (0.5-38.9 mmol/L).

QUESTIONABLE OR INCONSISTENT RESULTS

If test results are unusual or inconsistent with clinical signs and animal condition:

- Make sure the confirmation window of your test strip is completely filled with blood.
- Check the expiration date of your test strips.

CHEMICAL COMPONENTS

Glucose dehydrogenase (<i>E. coli</i>)	8%
Electron shuttle	55%
Enzyme protector	8%
Non-reactive ingredients	29%

ADDITIONAL INFORMATION

Always wear gloves and follow your facility's biohazard control policy and procedures when performing tests involving animal blood samples. Use fresh whole blood samples only. Veterinarians may use test strips to test capillary and venous blood sample.

Sample Size:	0.8 µL
Reaction Time:	5 seconds
System Measurement	
Range:	10 to 700 mg/dL (0.5-38.9 mmol/L)
Hematocrit Range:	0% to 70%

ACCURACY

Table 1 Accuracy results for glucose concentration < 100 mg/dL (5.55 mmol/L)

Within ±5 mg/dL (Within ±0.28 mmol/L)	Within ±10 mg/dL (Within ±0.55 mmol/L)	Within ±15 mg/dL (Within ±0.83 mmol/L)
94.5% (312/330)	100% (330/330)	100% (330/330)

Table 2 Accuracy results for glucose concentration ≥100 mg/dL (5.55 mmol/L)

Within ±5%	Within ±10%	Within ±15%
78.9% (497/630)	99.0% (624/630)	100% (630/630)

Table 3 Accuracy results for glucose concentrations between 45.0 mg/dL (2.50 mmol/L) to 680.0 mg/dL (37.78 mmol/L)

Within ±15 mg/dL or ±15% (Within ±0.83 mmol/L or ±15%)
100% (960/960)

Precision

The CV (%) is less than 5% both in intermediate precision and repeatability.

SYMBOL INFORMATION

Symbol	Referent	Symbol	Referent
	Consult instructions for use		Manufacturer
	Temperature limitation		Do not reuse
	Use by		CE mark
	Batch code		Humidity limitation

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